Powersurge

IMPLANTOLOGY CONTRA ANGLE 20 :1
CONTRE ANGLE D'IMPLANTOLOGIE 20 :1
IMPLANTOLOGIE WINKELSTÜCK 20 :1
CONTRAANGULO PARA IMPLANTOLOGIA 20 :1
CONTRAANGOLO D'IMPLANTOLOGIA 20 :1
CONTRA-ÂNGULO PARA IMPLANTOLOGIA 20:1

PS20 - PS20P

INSTRUCTIONS FOR USE - NOTICE D'INSTRUCTION
NOTA INFORMATIVA DE INSTRUCCIÓN - FOGLIO D'ISTRUZIONI
GEBRAUCHSANWEISUNG - INSTRUÇÕES DE USO

CONTENTS

1. MEANING OF SYMBOLS USED
2. SPHERE OF APPLICATION
3. GENERAL SAFETY INFORMATION
4. TECHNICAL CHARACTERISTICS
5. USING THE CONTRA-ANGLE
6. HYGIENE AND MAINTENANCE
7. REPAIRS
8. ACCESSORIES
9. GUARANTEES
10. CONDITIONS OF STORAGE & TRANSPORT
11. DISPOSAL OF PRODUCT

1. MEANING OF SYMBOLS USED

WARNING SIGNALS :

ATTENTION :
Important comments !

Not sterile

COMMENTS :

General indications
without danger

Please read !

OTHER LOGOS :

Visual examination

Autoclave at 135°C

Heat disinfectable

Reference

Serial number

Manufacturer

2. SPHERE OF APPLICATION

Contra-angles are intended for use in dental care within the field of dental surgery (drilling, tapping of cavities intended to receive dental implants, placement of dental implants; placement of prostheses). All forms of misuse are prohibited and could be dangerous.

Directive applied :

Community Directive 93/42/CEE, transposed by French regulations concerning medical devices (Official Bulletin L169, 12 July 1993) as amended by the 2007/47/CEE directive.

According to these instructions, the contra-angle may only be used by a user with experience in dental medicine, for the application described and with respect for current provisions concerning the prevention of accidents at work and the indications of the present instructions for use. This medical device must be prepared and maintained exclusively by persons with training in the field of protection against infections, self-protection and patient protection.

According to these instructions, the user is obliged to :

- Use only fault-free work instruments.
- Use the contra-angle only on drive units that comply with the directives of Standard EN 60 601.
- Observe the aims of correct use.
- Protect him/herself, the patient and third parties against all dangers.
- Avoid all contamination through the product.

3. GENERAL SAFETY INFORMATION

Before use, check that the contra-angle is not damaged and that no parts are missing.

- Use gloves, protective goggles and a mask.
- When inserting or changing the rotating instrument, ensure that the contra-angle is properly stopped.
- After inserting a rotating instrument, verify proper resistance with slight axial traction.
- Do not handle the rotating instrument while it is working.
- Activate the clamping system only when the motor is switched off.
- Connect or disconnect the contra-angle to the motor only when the motor is switched off.
- Pressing the push-button while the instrument is working can overheat the rotor and cause the rotating instrument to be lost.
- Ensure that the cooling flow is always sufficient, regular and properly orientated (risk of bone necrosis).
- If the tapping or screwing torque is too high the risk of bone necrosis or fissure increases, as does the risk of damage to the implant or instrument. Respect the implant manufacturer's instructions.

If there is visible damage or malfunction, stop using the instrument immediately and contact your approved distributor or B.A. International.

EN

FR

IT

DE

ES

PO

A poorly maintained instrument may overheat and cause unusually severe burns.

If you have any questions on the device, also contact your approved distributor or B.A. International. No alterations or additions to the device should be made without the express consent of B.A. International.

Only use accessories provided for this device. Do not use accessories for this contra-angle on any other type or make of contra-angle or on any other device.

4. TECHNICAL CHARACTERISTICS

(*) Values for information.

If a longer and wider rotating instrument is used, the user is responsible for choosing the right operating conditions to prevent any risks to patients or third parties.

	PS20	PS20P
Rate of reduction / multiplication	20:1	
Color	Green	
Adjustable stop system	Non	Non
Weight (g)	84	84
Tool fixing system	w/ latch	w/ push button
Motor connection standard	ISO 3964	
Maximum motor speed (rpm)	40 000	
Maximum torque for motor N/cm	7	
Max. torque for instrument N/cm	50	55
Tool type as per NF EN ISO 1797-1	Type 1	
Diameter of tools (mm) as per NF EN ISO 1797-1	2,35	
Max. length recommended by B.A. (mm)	50*	
Max. diameter of active tool part, recommended by B.A. (mm)	10*	
Spray water flow (ml/mm)	> 50	

5. USING THE CONTRA-ANGLE

The contra-angle is supplied **non-lubricated** and **non-sterile**. Before the first use, the contra-angle must be **lubricated** and **sterilised**. (see section on "hygiene and maintenance").

Before using, ensure that the contra-angle is not damaged and that no parts are missing.

Connection to the motor

ATTENTION : The contra-angle should only be connected to the motor when the motor is switched off.

Connect the contra-angle on the motor coupling until a click is heard. To do this, keep the motor and contra-angle in the same axis

For some motors, the side button must be activated to make the connection. Pull slightly on the contra-angle to verify before each use that it is correctly engaged with the motor coupling.

Functional test

Start the motor.

NOTE : If you notice heating, irregularity, vibration or unusual noise while the contra-angle is operating, contact your approved technical department immediately.

Disconnecting the motor

The contra-angle must be disconnected only when the motor is stopped.

Remove the contra-angle, keeping it within the axis of the motor.

NOTE : If the contra-angle is not used for long periods, do not leave it connected to the motor otherwise a leak of oil may occur in the motor.

Connecting / disconnecting the rotating instrument

For all tool-handling processes, protective gloves should be worn (risk of cuts).

Verify the condition of the rotating instruments used and handle them with due care and precaution.

Do not use a worn instrument (risk of necrosis and consequent rejection of implant).

Whenever the instrument is inserted or removed, the motor must be switched off.

Introducing and clamping the rotating instrument :

→ Push-button : exert pressure, using the thumb, on the center of the push-button.

→ Latch type : allow the latch to pivot.

→ Position the instrument by turning it gently, until it engages, then release the push-button or close the latch.

→ Ensure the rotating instrument is holding properly by means of slight axial movement after each change of tool.

Adjusting the stop

The stop must be adjusted outside of the mouth cavity.

→ Pivot the latch and bring the stop to the desired position. Close the latch.

Check that the stop is holding properly after each adjustment and each change of tool by exercising traction.

Check the position of the stop properly after each adjustment and after each change of instrument; use the ruler.

Do not use the stop for tapping operations : this may damage the bone or the device if the drive motor does not stop early enough.

6. HYGIENE AND MAINTENANCE

Re-useable medical devices must be re-sterilised by staff with proper training and protection, in accordance with current regulations.

The re-sterilisation protocol must be adapted according to the risk of infection.

GENERAL RULES

- Refer to the manufacturer's instructions for each product used. In particular, respect the concentrations, exposure times, solution renewal processes and product life span.
- Use only products intended for the maintenance of medical and surgical equipment.
- Prohibit antiseptics intended for use on the skin or mucous membranes. Use only products with a neutral or slightly alkaline pH. Prohibit products that contain aldehydes, alcohols or other products likely to fix proteins.
- Use only products that are compatible with stainless steels (no chlorine compounds).
- Wear suitable protective clothing. To avoid any risk of infection or injury, protective gloves must be worn at all times.
- Never immerse the contra-angle in any solution. Never clean the contra-angle in an ultrasonic tank.
- All handling processes involving contra-angles must be conducted with the motor disconnected or switched off.
- Never mix products. Respect the instructions concerning disposal of used products.
- Never immerse the contra-angles in disinfectant baths or in an ultrasonic tank.

Head assembly/disassembly

- Disconnect the contra-angle from the motor : remove the rotating instrument and the tubes.
- Using the key supplied, free the head nut.
- Remove the head.
- Remove the collar cartridge.

Reinstalling : proceed in the reverse order

- Re-tighten the head screw properly.
- Check that the head is holding by applying axial traction.

Pre-disinfectionn

To be carried out immediately after the surgery.

- By vaporisation, or using wipes containing microbiologically controlled detergents or disinfectants.

Do not immerse the contra-angles in disinfectant baths or in an ultrasonic tank.

Cleaning

- Rinse and brush each element with demineralised water.
- Clean the irrigation channels using the reaming thread.

Risk of injury while using the reaming thread : wear protective gloves and goggles.

- Dry (absorbent cloth or compressed air).

Disinfection

- Using disinfectant wipes (recommended) or vaporisation.

Cleaning and disinfection may be carried out in a heat-disinfecting washer.

Respect the instructions of the cleaning product appliance manufacturer.

Check that the correct rinsing and drying processes are used.

Lubrication

→ Lubricate after each decontamination process and before each sterilisation.

- Keep away from all sources of ignition.

In particular : do not smoke !

- Wear a facial protection mask.
- Remove the rotating instrument.
- Introduce the spray nozzle behind the contra-angle.
- Check that the end of the connection is the one to be used for contra-angles.
- Cover the head of the contra-angle with a wipe.

- Point the head of the contra-angle downwards.
- Vaporise several times, holding the contra-angle firmly, until the liquid that comes out is clear.
- Wipe away excess oil with a soft cloth or wipe.

Function test

This test must be carried out prior to each sterilisation.

Connect the contra-angle to the motor: point the head of the contra-angle downwards. Turn at slow speed for 30 seconds to remove any excess oil, then at full speed for 30 seconds. Wipe the contra-angle with a disinfectant wipe if oil has flowed out.

Note: If you notice heating, irregularity, vibration or unusual noise while the contra-angle is operating, contact your approved technical department immediately.

Sterilisation

→ Only sterilise instruments that have been pre-disinfected, cleaned, decontaminated, lubricated and tested.

The contra-angle is delivered non-sterilised. It must be sterilised before the first use, and after each use.

The device can be sterilised at 135°C at 2.13 bars (275°F at 30.88 PSI), in a steam autoclave for 20 minutes minimum (sterilisation maintenance time).

- We strongly recommend the use of a category B autoclave.
- All other methods of sterilisation should be prohibited.

Read the autoclave manufacturer's instructions for use.

→ Use the sterilisation bags adapted for the contra-angle and the autoclave, in accordance with current standards. Always one contra-angle per bags only.

→ Respect the space between the bags and do not overload the autoclave.

→ Check that the device does not contain any corrosion point or crack and ensure it is working properly. Ensure that the product is dry. If necessary, dry any water residue with pressurised air of medical quality.

→ After each sterilisation cycle, check that there is no residual water inside or outside the packaging. Check that the color change for the passage indicator is correct.

→ In order to avoid water retention, position the bag in the autoclave so that the hollow parts are directed downwards.

→ Keep the devices under sterilisation bags away from light, damp and all forms of contamination. Follow the packaging manufacturer's recommendations.

→ After sterilisation, the device must not be stored longer than 1 month. Label the devices, specifying the expiration date. After the expiration date, start the cleaning and sterilisation cycle again.

7. REPAIRS

In the event of breakdown, contact your approved distributor or our after-sales service directly.

All repairs must be carried out using manufacturer-certified parts and subsets. Repairs must be carried out only by the technical service of an approved dealer or by B.A. International service center.

For all overhauls or repairs, the device must be returned complete, clean and sterile, with proof of sterilisation. It must be accompanied with a document describing the problem encountered and containing the full details of the user practitioner.

For warranty requests to be taken into consideration, please send a copy of the invoice or delivery sheet with the device.

For all overhauls or repairs, please send the complete contra-angle. Renewal of spare parts is guaranteed for 7 years after marketing of the product stops.

8. ACCESSORIES

Order from your approved distributor.

Description

Internal irrigation system

Sterilisable metallic Y-connection

Head removal key

B.A. spray oil

Spray release metallic thread

Ref.

BA690025

BA690015

BA690028

BAOIL

BA690029

9. WARRANTY

This device is guaranteed for parts and labour against all manufacturing defects for a period of 12 months from date of invoice. This guarantee does not apply to perishable parts or cover transport costs.

Carrying out an operation under guarantee does not extend the duration of the guarantee.

For warranty requests to be taken into consideration, please return a copy of the invoice or the delivery sheet with the device.

All alterations or additions to the product, made without the express consent of B.A. International, will invalidate the guarantee.

EN

FR

IT


DE

ES

PO

B.A. International cannot be held responsible for damage or subsequent events arising or likely to arise through normal wear and tear, incorrect use, cleaning or maintenance, non-observation of instructions relating to use or connection, clogging or corrosion, impurities in the air or water supply system, unusual chemical or electrical effects, or non-compliance with instructions for use, maintenance or mounting from B.A. International or other manufacturer's instructions.

10. CONDITIONS OF STORAGE AND TRANSPORT

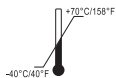11. DISPOSAL OF PRODUCT

At current knowledge levels, the product does not contain any environmentally harmful substance.

The product must be sterilised before being disposed of.

Observe the national regulations concerning the discarding of medical and surgical devices and instruments.

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