Maxima® ELITE

by HENRY SCHEIN®

INSTRUCTIONS FOR USE

Read before use. Keep in safe place for future reference.

⚠ Caution: Federal (US) law restricts this device to sale by or on the order of a licensed professional.

112-5681 Maxima Elite 1:5
112-5813 Maxima Mini Elite 1:5
112-6926 Maxima Elite 1:1
112-6927 Maxima Elite S

⚠ Warning

- If the handpiece is defective or experiences mechanical failure, handpiece may heat up and result in burn injury to patient.
- Do not press push button end cap during use since this could result in overheating of handpiece head. Special caution must be exercised during use to keep cheek tissue away from push button end cap. Contact with cheek tissue may cause the push button to depress and result in burn injury to patient.
- Before each use, check for vibration, noise, and overheating. If any abnormalities are observed, stop using handpiece immediately and contact Henry Schein® ProRepair®.
- Never operate the handpiece without cooling air and water spray activated. Cooling air is emitted by ISO motor coupling and must be fully activated. Operate only with filtered water (< 50 µm). Clean nozzles regularly.

⚠ Caution

- Do not run handpiece without a bur fully inserted and locked in chuck.
- Do not operate handpiece with a bur partially inserted, or extended out.
- Do not disassemble, modify or alter handpiece.
- Never attempt to remove bur while running handpiece.
- Sterilization is required before first use.
- Do not drop, hit or subject to excessive shock.
- Purge water and air channels for 20 - 30 seconds before each use.

	1:5 (112-5681)	1:5 (112-5813)	1:1 (112-6926)	1:1 straight (112-6927)
Gear ratio	1:5 red	1:5 red	1:1 blue	1:1 blue
Maximum motor speed	40,000 rpm	40,000 rpm	40,000 rpm	40,000 rpm
Maximum operating speed	200,000 rpm	200,000 rpm	40,000 rpm	40,000 rpm
Light	Cellular glass rod optics			
Cooling	4 spray ports	1 spray port	1 spray port	1 spray port
Head height	14.6 mm	13.4 mm	12.4 mm	-
Head diameter	8.7 mm	8.7 mm	8.7 mm	-
Bur type (ISO 1797-1)	Friction grip (FG) Type 3	Friction grip (FG) Type 3	Latch-type (RA) Type 1	Nosecone (HP) Type 2
Weight	66 g	64 g	64 g	85 g

ELECTRIC HANDPIECE ADVISORY

The FDA has issued an advisory regarding the potential for patient injury due to improperly maintained or failing speed increasing handpiece attachments (red). Friction in the internal bearings can cause overheating and possibly burn a patient. Minimize heat by following the maintenance and lubrication procedures found in these instructions and confirming that there is adequate air and water flow during use. Also, perform a Feel/Listen/Touch test (covered in the Operation section) before each procedure.

OPERATION

⚠ Warning

- Do not pull the patient's cheek back with the handpiece while the motor is running. This could actuate the push button and create a risk of burn.
- Purge water and air channels for 20 - 30 seconds before each use.

⚠ Caution

- Handpiece should only be fitted or removed when motor is not running.
- Never operate micromotor without an attachment to avoid injury from exposed drive shaft.

Before use

Before each use, perform this Feel/Listen/Touch testing. If your handpiece fails any of these tests, discontinue use immediately and send to Henry Schein® ProRepair® for service.

Feel - Insert test bur and manually rotate it between your fingers. You should FEEL the bur rotating freely.

Listen - Connect attachment, then turn on cooling air, water, and motor and gradually increase speed to maximum level (40K on control box panel). Run for 30 to 45 seconds without a load and LISTEN for unusual grinding or clicking noises when operating in forward and in reverse.

Touch - While running handpiece at full speed, TOUCH head and neck areas to check for excessive heat. It should feel cool to the touch.

Handpiece Connection (Fig. 1)

Attaching the handpiece

Insert the handpiece until it snaps into place. Handpiece should sit flush on motor.

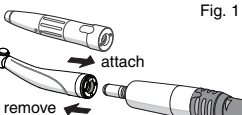
Fig. 1

Removing the handpiece

Pull off the instrument. Do not hold or pull on hose.

Note: In order to prevent oil from bleeding into the electric motor do not leave any instruments on the motor overnight. Never lubricate the electric motor.

Bur Types

⚠ Caution

- Only use bur types shown in the SPECIFICATIONS table.
- Do not use bent, irregularly worn or damaged burs.
- Always keep burs clean. Dirt and debris on the shank may cause clogging, loss of grip or premature wear of chuck.

Cleaning Push-Button Chuck (Fig. 4)

Note: It is recommended that the push-button chuck be periodically cleaned.

- Remove bur.
- Insert Henry Schein® Maxima HP Spray nozzle into chuck and spray for 1 - 2 seconds.
- Work chuck mechanism to distribute cleaner/lubricant by repeatedly pressing button.
- With push button depressed, insert and remove bur several times to break-up remaining debris.
- Wipe off excess cleaner/lubricant with a dry, lint-free cloth.

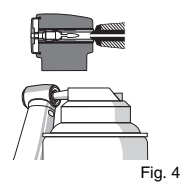
Fig. 4

Cleaning Water Spray Ports (Fig. 5)

Note: It is recommended that the spray ports be periodically cleaned.

- Carefully clean ports using cleaning wire supplied with handpiece.
- Connect handpiece and run with water flow activated to confirm ports have been fully cleared of debris.
- Repeat steps 1 and 2 as necessary.

Fig. 5

Cleaning Light Guide (Fig. 6)

- Clear light guide surfaces (A) of particulates by spraying with air syringe.
- Wipe light guide surfaces with a soft cloth or cotton swab dampened with alcohol.

Fig. 6

STERILIZATION

⚠ Warning

- Sterilize by autoclave after each patient use.
- Sterilization is required before first use.

⚠ Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
- Do not use ultrasonic bath, boiling water or chemicals to clean handpiece.
- Autoclave heating element could be located at bottom of the sterilization chamber. To ensure that handpieces do not overheat, place handpieces on central or upper tray.
- Do not autoclave handpieces together with other instruments.

- Wipe off handpiece with damp cloth.
- Place handpiece in sterilization pouch and seal. Bag handpieces individually.
- Autoclave with three times initial vacuum: at least 3 minutes at 134°C - 1°C / +4°C (273°F - 1.6°F / +7.4°F) / Drying time: 20min. Autoclave using the gravitation method: at least 10 minutes at 134°C - 1°C / +4°C (273°F - 1.6°F / +7.4°F) or at least 60 minutes at 121°C - 1°C / +4°C (250°F - 1.6°F / +7.4°F) / Drying time: 30min.

SERVICE

For items that require service, including warranty service, please send to Henry® Schein ProRepair®.

Henry Schein ProRepair
620 S. Placentia Ave.
Placentia, CA 92870 USA
Phone # 714-579-0175
Fax # 714-579-0186

Warranty

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 2 years from date of purchase.

Henry Schein® will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

Changing Burs (Fig. 2)

⚠ Caution

- Do not run the handpiece with bur not fully inserted or securely locked into place.
- Never run handpiece without a bur.
- Never attempt to remove bur while running handpiece.
- After bur is seated, pull on bur to make sure it is locked into place.

Inserting bur (Contra Angle)

- Press push button and insert bur into head until it stops.
- Release pressure on button; bur should now be secure.
- Firmly tug on bur to ensure that it is properly secured.


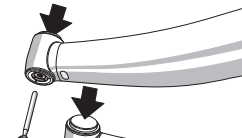
Fig. 2

Removing bur (Contra Angle)

- Press push button on end cap with thumb.
- Pull to remove bur.

Inserting and removing bur (Straight HP)

- Turn the center section in the direction of the arrow until it stops.
- Insert the bur completely, then turn center section back until dots align.
- To remove, repeat step 1 to open the chuck then remove bur.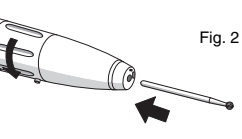

MAINTENANCE

⚠ Warning

- Failure to properly maintain handpiece may result in overheating and burns to patient.

⚠ Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves

Cleaning and Lubrication (Fig. 3)

Note: Cleaning and lubrication is required before each sterilization cycle.

- Remove bur.
- Detach handpiece from motor.
- Fit spray adapter (included with handpiece) onto universal cap nozzle of Henry Schein® Maxima HP Spray (112-6928).
- Insert instrument as far as it will go and hold it in place.
- Apply spray for 1-2 seconds, then run handpiece to purge cleaner/lubricant.
- Repeat steps 4 and 5 until cleaner/lubricant expelled from head is clear.
- Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

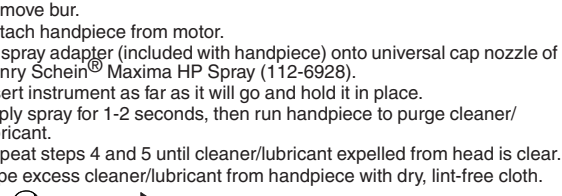
Fig. 3

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Henry Schein U.K. Holdings Ltd.