

◎ CARE BETTER, SMILE BRIGHTER
 IMPROVE WORKING LIFE OF YOUR HANDPIECE THROUGH EFFICIENT CLEANING AND LUBRI-
 CATION IN A PLEASSED WAY.

TTcare 100+

LUBRICATION UNIT



MODEL

TTcare 100+ 08090012

Dimensions	W295 X D295 X H385mm	
Weight	9kg/ 19.85lbs	
Electrical ratings	100-240V, 50-60Hz, 60W	
Protection class	I	
Ambient temperature	5-40°C (41-104°F)	
Sound level	<60 (65) dB (A)	
Relative humidity	max. 85%	
Compressed air	4-6 bar (58-87 ps)	
Air consumption	40 NL/min	
Compressed air quality	Residual oil content	max. 0.1 mg/m ³
	Residual dust	max. 1 µm ; max. 1 mg/m ³
	Residual water	max. 0.1 g/m ³ at -40°C (-40°F)

EASY TO USE

TTcare 100+ is designed to response to Dentists' requests. Its user-friendly interface simplifies the process of cleaning handpieces. It takes 15 sec only to lubricate and clean each handpiece, and less than one minute for three at a time. It's fast and time-saving to provide premium care for handpieces.

MINOR DETAILS MAKE BIG DIFFERENCES

In order to improve customer's using experience, TTcare 100+ includes the exceptional details: LCD screen, simply disassembled front door, exclusive chuck care system, easy spray can insertion and replacement, air driven motor design to highlight its functionality and features.

FEATURES

- Automatically cleans and maintains up to three handpieces
- Unique chuck care system
- Optional lubricant coverage throughout the handpiece
- Compressed-air connection: excess lubricant purging with special suction design
- Automated reproducible quality of care
- Easy spray can insertion and replacement
- Can coupling with unique safety valve
- Oil collecting drawer for excess lubricant
- LCD screen display
- Is capable to run without compressed air
- Maintain 1 handpiece in only 15 secs
- Unique drive motor design ensures the excellent clean process
- Quickly disassemble the front door and plain surface for easy clean process

- CARE BETTER, SMILE BRIGHTER
IMPROVE WORKING LIFE OF YOUR HANDPIECE THROUGH EFFICIENT
CLEANING AND LUBRICATION IN A PLEASD WAY.

EXCLUSIVE CHUCK CARE SYSTEM

- STEP 1** Place chuck into chuck care adapter.
- STEP 2** Press program button to start the processing.
- STEP 3** Clean and care handpiece's chuck.



DONE IN A SIMPLE WAY

- STEP 1** Flip down the door and snap one, two or three handpieces onto the adapters.
- STEP 2** Close the door.
- STEP 3** By pressing the corresponding buttons to start the automated, internal cleaning and care of up to three handpieces.
- STEP 4** Finish and remove the handpieces.



FEATURES OF ACTIVE-LUB SYSTEM


- Optimizing the overall lubrication: The Active-lub system evenly distributes the lubricants to the inside of all components to lubricate contra-angle handpieces.
- Reliability and flexibility: Driving motor uses the belt to drive the operation smoothly, which brings out the features of quiet, stable, and vibration-free. The driven ports could be optionally increased or decreased based on the need.
(For contra-angle only)



Simply Disassemble

With thoughtful idea, it's effortless to keep TTcare 100+ in a good stage.

Simply disassemble the front door for further cleaning.

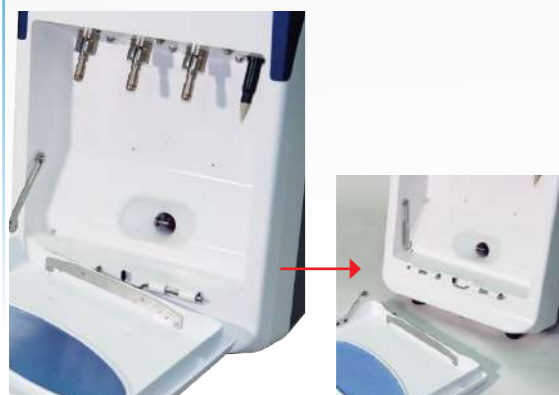

Amazed by fancy design

Compressed air purge excess lubricant into suction hole and oil collecting drawer, that makes inside of TTcare 100+ to stay clean.



TTlube Spray 08090011

Easy spray can insertion and replacement.



HANDPIECE ADAPTOR AVAILABILITY

Image	Description	Product Code	Image	Description	Product Code
	Adapter KaVo	OPL0345		Adapter Star	OPL0413
	Adapter W&H	OPL0346		Adapter Midwest	OPL0551
	Adapter NSK	OPL0344		Adapter Midwest CA	OPL0583
	Adapter Bien-Air	OPL0348		Adapter Borden	OPL0110
	Adapter SIRONA	OPL0347		Blister	OPL0350
	Adapter E-type	OPL0349		Absorption pads(set)	OPL0351